**PalaVeneer® 6/PalaVeneer® 8**

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d’emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l’uso
- PT** Instruções de uso
- NL** Gebruiksaanwijzing
- SE** Bruksanvisning
- DK** Brugervejledning
- NO** Bruksanvisning
- FI** Käyttöohjeet

## PalaVeneer® 6/PalaVeneer® 8

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**Produktbeschreibung**

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**Instructions for use** **GB**

**Product description**

Veneers for dental prosthetics, PalaVeneer 6 is used in the anterior region and PalaVeneer 8 in the posterior region to produce permanent and removable dental prostheses. They are produced from the NanoPearls material and can thus be combined with all Pala Mix&Match teeth.

**Composition**

Cross-linked PMMA copolymer, inorganic fillers, pigments

**Indications**

For the production of veneers for

- Telescopic work
- Model cast prosthetics
- Implant structures
- Cover dentures
- Aesthetic try-ons
- CAD/CAM bridges and bars
- Laboratory manufactured temporary restorations

**Contraindications**

Direct adhesion to natural tooth tissue in the mouth is contraindicated. Only for processing in dental laboratories in accordance with intended use.

**Notes**

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer. When processing the PalaVeneer veneers with rotating instruments, please observe the usual safety measures, such as suction and a protective surgical mask.

**Processing**

**Set-up**

Adapt the PalaVeneer veneers to the framework structure. Set up the PalaVeneer veneers with soft, dentine-coloured wax to still allow repositioning. Once the set-up is complete, fill cervical, approximal and palatal/lingual cavities with harder wax and perform the wax up.

**Producing the matrix**

Create a matrix from workable silicone impression material with an increased hardness (Shore A > 60). Remove the silicone matrix once it has cured.

**Conditioning the metal framework**

We recommend mechanical retentions on the metal framework. Clean the metal framework, sandblast it (110–125 µm Al<sub>2</sub>O<sub>3</sub>, 3 bar), remove dust with oil-

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	<b>KULZER</b> MITSUI CHEMICALS GROUP
	<b>Manufacturer:</b> Kulzer GmbH Leipziger Straße 2 63450 Hanau (Germany)
	<b>Distributed in USA / Canada exclusively by:</b> Kulzer, LLC 4315 South Lafayette Blvd. South Bend, IN 46614-2517 1-800-431-1785
	<b>Caution:</b> Federal law restricts this device to sale by or on the order of a dental professional.
	<b>Importado e Distribuído por</b> Kulzer South América Ltda. CNPJ 48.708.010/0001-02 Rua Cenzo Sbrighi, 27 – cj. 42 São Paulo – SP – CEP 05036-010 sac@kulzer-dental.com Resp. Técnica: Dra. Regiane Marton – CRO 70.705 Nº ANVISA: vide embalagem
	<b>CE 0197</b>

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Stand: 2014-08

free compressed air; do not steam off. Condition metal framework with Signum® metal bond I and II (see Signum® metal bond I and II instructions for use). Cover the metal framework with 2-3 thin layers of light-curing Signum® opaque F and cure each layer individually. For an optimal bond, apply the opaquer without undue delay and without contaminating the surface by touching it. Then apply a tooth-coloured, PMMA-based acrylic made of powder and liquid.

**Conditioning**

Remove the PalaVeneer veneers, steam off wax residues, sandblast the contact surfaces of the veneers to the acrylic (110–125 µm Al<sub>2</sub>O<sub>3</sub>, 2 bar) or roughen with a coarse diamond. Remove dust with oil-free compressed air; do not steam off. Affix the veneers in the silicone matrix. To improve the adhesion between the acrylic and the veneers, all the contact areas of the veneers against the acrylic should be coated with the Palabond® adhesive system. Apply Palabond® twice with a brush (no metal ferrule) and leave for 30 sec after each coat. After the second coat, the adhesion promoter remains active for approx. 10 min.

**Model preparation**

After removal of the wax with hot water (no chemical additives), the plaster model is thinly coated twice with Aislar® (see Aislar® Instructions for use). This ensures perfect surface insulation between the model and the acrylic.

**Completion**

Please observe the instructions for use of the acrylic during production. Using the pre-press technique, apply the acrylic to the luted veneers with a slight excess and then pre-press and affix the silicone matrix in place. Remove any excesses and perform the wax up of the palatal/lingual surfaces.

**Polymerisation**

The polymerisation parameters can be found in the instructions for use of the acrylic.

**Trimming and polishing**

The PalaVeneer veneers can be trimmed with a tungsten carbide bur, lenticular silicone polisher, separating disc, goat hair brush with acrylic polishing paste and a cotton wool buffer with high-lustre polish, for example.

**Storage advice**

In order to avoid melting the wax, the PalaVeneer veneers should not be stored at temperatures over 28°C (82°F) for prolonged periods. Even if the wax becomes slightly soft, this has no adverse effects on the quality of the product. Keep out of direct sunlight.

**Note**

For disposal, please follow the instructions on the material safety data sheet or national regulations.

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® = registered trademark of Kulzer GmbH
Ⓢ = registreret varemærke tilhørende Kulzer GmbH
Ⓢ = registrert varemærke for Kulzer GmbH