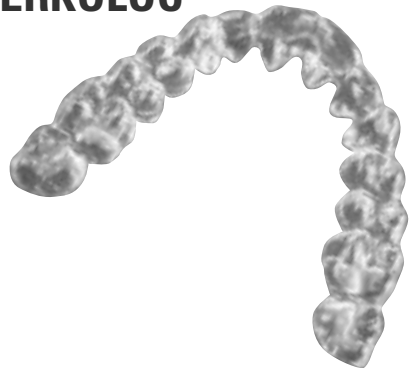
# ERKOLOC

DIN EN ISO  
13485: 2003

Erkoform-3/3d  
Erkopress 300 Tp/-ci  
  
 1,8 mm, 135 °C  
 3,0 mm, 120 °C  
  
 1,8 mm, 1:15 min  
 3,0 mm, 1:45 min

Erkoform-RVE  
Erkopress 200 E  
  
 1,8 mm, 1:20 min  
 3,0 mm, 2:05 min

Lagerung • storage • stockage

ERKODENT Erich Kopp GmbH

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72285 Pfalzgrafenweiler, Germany

Tel. + 49 (0) 74 45/85 01-0

Fax + 49 (0) 74 45/20 92

info@erkodent.com • www.erkodent.com

# DE ERKOLOC

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# EN ERKOLOC

## Chemical characteristics:

SBS, hard side: styrol-butadien-styrol  
EVA, soft side: ethyl-vinyl-acetate  
PE, insulating foil: polyethylene  
Harmless to health, BgVV (Federal Health Office)  
listed thermoplasts with tested biocompatibility.  
Insoluble in water, inactive, harmless to ground  
water.

## Disposal/Recycling:

Household waste (combustible, low in pollutants and  
high in energy)/general recycling for plastic.

**Field of use:** (see brochure Thermoforming tech-  
nique and material card) Comfortable stabilization  
and occlusal splints.

## Material properties:

Double layer plate, hard/soft. Hard layer adjustable  
by addition using acrylics (Resilit-S, 817 501 +  
817 503). Do not move splint for app. 2 hours after  
building-up with acrylics to ensure a stable bond.  
Limited durability, not suitable for permanent splints  
(recommendation for permanent splints:  
ERKOLOC-pro).

## Finishing:

(see brochure Thermoforming technique) Finishing  
set Quick 2 (110 877). The spacer foil will be taken  
off only after thermoforming and finishing.

## Plastification:

(for machines without heating time control) The hard  
side of the ERKOLOC plate should be heated. Place  
the foil in the devices in such a way that the spacer/  
insulating foil is pointing towards the model. Test  
softness of foil with an instrument. If permanent  
impressions result, thermoform.

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